

## QUICKTIP

# Music Program

The Music Program is a dedicated program designed to enhance sound quality for patients who spend time listening to music.\*

## The dedicated music program includes:

- Fitting rationale for music, focusing on providing audibility for soft music and transparent sound quality for loud music
- Pro Fit software adjustments tailored for music

Fitting and evaluation of the Music program are subjective. Make adjustments based on patient feedback regarding music sound quality. It is recommended to play music at the patient's preferred listening level while adjusting the Music Program for best fitting results. Select the Media Player in the **Tools** tab in the top tool bar, under the Category Name, then Music, then select desired Audio File, or play music from patient's personal music device.

- 1 Launch Pro Fit and connect hearing devices.

*Refer to the Fitting Protocol QuickTips for more information.*

- 2 Select **QuickFit** from the Fitting session or from the **Flyout Menu** ≡.

## Music Program

- 1 Select **program 2, 3, and/or 4**, select the **arrow** to extend the programs menu, then select **Music** to program the Music Program in one or more of the program slots.

- 2 Select the Music EQ Controls on the QuickFit screen to adjust the Music Program to the patient's preference. Adjustments include: [Fig. 1]

- All Hz
- Bass
- Mid
- Treble

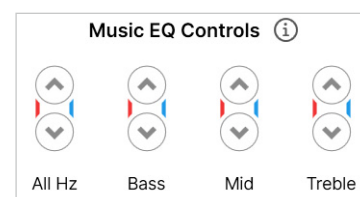


Figure 1

- 3 Select **Fine Tuning** from the Follow-Up session or from the **Flyout Menu** ≡ or from the QuickFit screen to access more channel-specific and input level-specific adjustments, if desired.
- 4 Reference Loud and Soft Music Equalizers to view Music Program settings. [Fig. 2]
- 5 Select **+ Bass, Mid, and/or Treble** to expand channel ranges.
- 6 Select channels and input levels to adjust, then select **up** and/or **down arrows**.

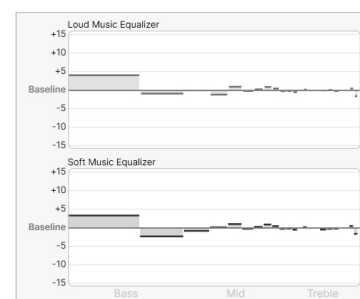


Figure 2

\*Available on all wireless and select non-wireless hearing aids. Contact your rep for specific information.