

QUICKTIP

Sensitive Compartmented Information Facility (SCIF) - Compliant Hearing Aids

For Professionals

Sensitive Compartmented Information Facility (SCIF)-compliant wireless hearing aids have all hands-free audio features and two-way audio features disabled. This solution prevents microphones from transmitting sounds from inside a Sensitive Compartmented Information Facility. Non-wireless products are inherently SCIF compliant.

SCIF Mode disabled features:


- 2-Way Audio phone calls
- Mobile app features:
 - Smart Assistant
 - Translate
 - Transcribe

NOTE: SCIF Mode will still allow use of the hearing aid mobile app and audio streaming from an accessory or smart device.

SCIF Mode




Edge AI Hearing Aids: Any wireless Edge AI device can be made SCIF compliant in the Pro Fit software.

To enable/disable SCIF Mode in Pro Fit:

- 1 Select **Tools** and **SCIF Mode** [Fig. 1]
- 2 Utilize the the **toggle switch** to turn SCIF Mode on or off. [Figs. 2 & 3]
- 3 If needed, print the **SCIF Compliance Letter** to verify compliance. [Fig. 2]
- 4 The Left Menu Bar will indicate if SCIF Mode is on . Select **SCIF** or **Tools** and **SCIF Mode** to open the SCIF Mode screen and change the setting. [Fig. 1]

Genesis AI Hearing Aids: SCIF is only available for the Genesis AI 24 RIC RT (SCIF) and RIC CROS RT (SCIF). These devices must be ordered as SCIF devices, as they have different part numbers*.

To disable/enable SCIF Mode in Pro Fit:

- 1 Expand the **Flyout Menu**  or utilize the **Left Menu Bar** and select **SCIF**. [Fig. 1]
- 2 Utilize the the **toggle switch** to turn SCIF Mode off or on. [Figs. 2 and 3]
- 3 If needed, print the **SCIF Compliance Letter** to verify compliance. [Fig. 2]
- 4 The Left Menu Bar will indicate if SCIF Mode is on  or off . Select **SCIF** to open the SCIF Mode screen and change the setting. [Fig. 1]

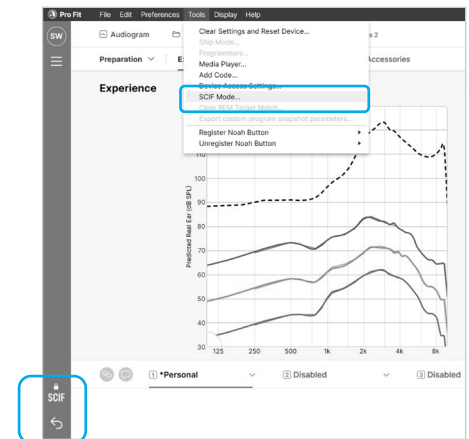


Figure 1

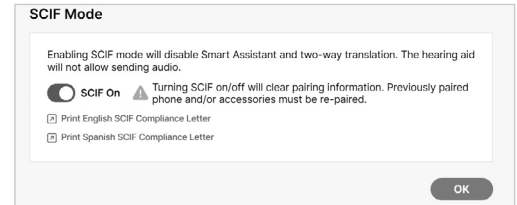


Figure 2



Figure 3

*If ordering through the Veterans Administration, orders must be placed through Prosthetics.