

Auracast assistant



Connect anywhere with Auracast assistant

Stay connected wherever you go! Easily find and join Auracast™ broadcasts—whether you're at the airport, the gym, or listening to public announcements. Enjoy instant, clear audio streamed directly to your hearing aids from the My Starkey app.

OMEGA^{AI}
EDGE^{AI}



Where to find it



My Starkey mobile app*

*Requires Hearing Aid Firmware version 10.3.1.5 or later.

Connecting to Auracast broadcasts through the My Starkey mobile app

1. From the Home screen, tap the **accessories icon**.
2. Tap **Auracast™ broadcasts**.
3. Tap the broadcast you would like to join.
4. Experience seamless audio streaming to your compatible Starkey hearing aids.

*Scan to check
device compatibility*

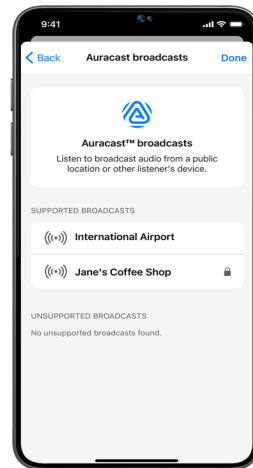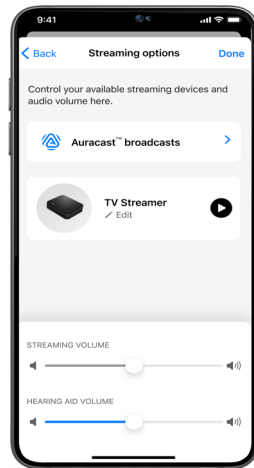

The Auracast™ word mark and logos are trademarks owned by the Bluetooth SIG. Any use of such marks by Starkey is under license.

Starkey logo, Starkey Edge, and Omega are trademarks of Starkey Laboratories, Inc.

©2026 Starkey Laboratories Inc. All Rights Reserved. P00017538 1/26 CARD4162-00-EN-ST