

QUICKTIP

CROS System Initial Fit Protocol with Pro Fit

Our wireless CROS System takes audio from an ear-level microphone and wirelessly transmits it to the opposite ear via Near-Field Magnetic Induction (NFMI) technology, optimizing patients' hearing experience. The CROS System is available for the following styles: RIC RT and RIC 312.

Pro Fit will guide the hearing care professional through three fitting session types: Preparation, Fitting, and Follow-Up.

Wireless CROS System is Composed of:

Transmitter

- Worn on the unaidable ear
- Microphone enabled for CROS and BiCROS

Receiver

- Worn on the normal ear or better hearing ear
- Microphone enabled for BiCROS
- CROS-enabled hearing aids will become a Receiver when brought into a fitting session with a Transmitter

NOTE: *The Transmitter and Receiver style must match (e.g., RIC 312 on both sides).*

Wireless Initial Setup

NOTE: *To power on rechargeable 2.4 GHz RIC hearing aids for the first time, briefly insert them into the appropriate charger for 3 seconds then remove.*

- 1 2.4 GHz hearing aids can only be programmed wirelessly via the 2.4 GHz or NOAHlink Wireless 1.0 or 2.0 programmer. Rechargeable hearing aids will ship with enough battery life to complete initial hearing aid programming, but not enough for a full day of use. Place hearing aids in the charger prior to the initial fitting to ensure there is enough battery life for use.

NOTE: *If the hearing aid battery charge is too low, a low battery warning will appear when reading devices. If this occurs, the fitting session can proceed, but a firmware update (if available) cannot. A turbo charge using the StarLink Chargers is enough to proceed with a firmware update.*

- 2 Attach SnapFit 2.0 receivers to RIC hearing aids.
- 3 If programming the Zn-Air RIC 312, insert batteries into the hearing aids.

Pro Fit Minute Fit®

The 2.4 GHz wireless CROS Receiver and Transmitter must be paired and programmed together at the initial fitting. Receivers can be programmed with or without the Transmitter present after the initial fitting. Streaming from the Transmitter is available during the fitting session.

- 1 Select the Receiver and Transmitter to be programmed. [Fig. 1]
- 2 Select **Connect** to begin a new fitting or for a follow-up fitting session using the current settings in the hearing aids.
- 3 Review and verify selections. Select **Apply Best Fit**. [Fig. 2]

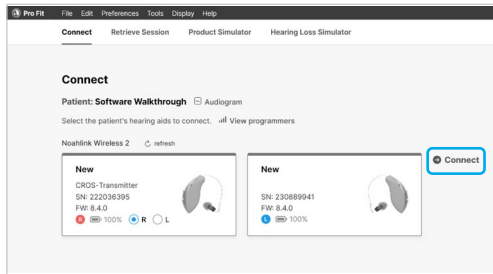


Figure 1

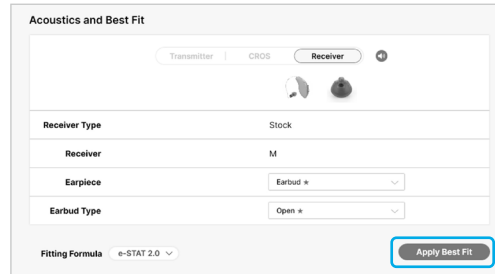


Figure 2

Preparation

Pro Fit advances to the Preparation session automatically when programming new hearing aids. [Fig. 3]

- 1 Set up new hearing aids.

A. Assign the Experience Level.

B. CROS and BiCROS modes can be enabled on a per-program basis. Select Programs and enable hearing aid programs, if desired. For each compatible program, select **Hearing Aid Only, CROS, or BiCROS.**

NOTE: Some programs are incompatible with the CROS/BiCROS Transmitter including: Telephone, Telecoil, Autophone and Autocoil.

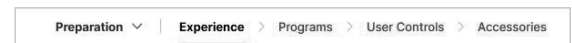


Figure 3

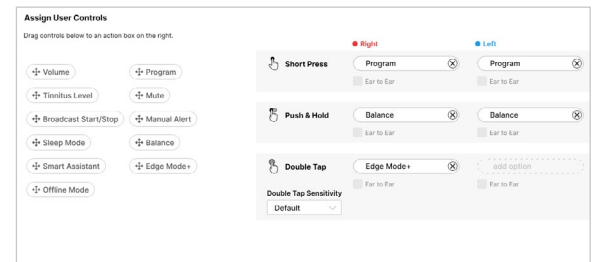


Figure 4

C. Select User Controls. Drag any user control you wish to enable from the left and drop it on a gesture on the right. [Fig. 4]

NOTE: Rechargeable hearing aids do not have battery doors. To provide patients with the ability to power off their hearing aids while not in use, the Push & Hold user control will be configured as "Sleep Mode" when shipped.

NOTE: Balance Control is unique to BiCROS fittings. BiCROS Balance Control adjusts the level of sound coming from the Transmitter relative to the Receiver. Changes will not affect the hearing aid response.

NOTE: Double Tap is available for a CROS system and will work on the Receiver side only. Double Tap Sensitivity may be adjusted by selecting the dropdown arrow under the user control options.

Preparation (Continued)

D. Select **Accessories** to pair and configure accessories, if desired. See **pairing information** for information on the pairing process. Once paired, adjust the **Hearing aid mic while streaming, Streaming sound** for the streaming accessories and the **Favorite Button** configuration for the Remote. Adjust the **Hearing aid mic while streaming phone** and **Streaming sound** for phone use.

NOTE: Audio from all 2.4 GHz wireless accessories will stream to the Receiver side only and the Transmitter side will be unavailable. Set the TV Streamer to monaural streaming for CROS/BiCROS.

- Utilize the **Flyout Menu** ≡ and the **Left Menu Bar** to access additional screens and help features. [Fig. 5]

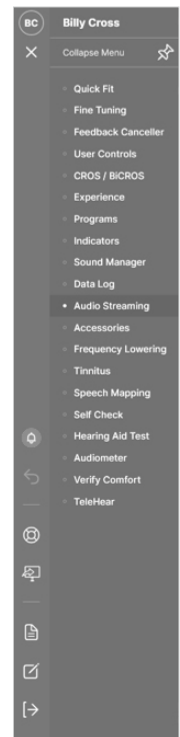


Figure 5

Fitting

Following the Preparation session, Pro Fit will advance to the Fitting session automatically. [Fig. 6]



Figure 6

- Insert the hearing aids into the patient's ears, reduce all sources of noise, then select **Begin Test** to run the **Feedback Canceller** initialization.
- If the hearing aid model is updated based on measurements from the feedback canceller test, choose **Target Match/Yes** to accept the measured response or choose **Do Not Target Match/No** to continue with the initial predicted response. [Fig. 7a/7b]

NOTE: The model is updated based on measurements obtained during the feedback canceller test. The updated model reflects the measured acoustics of the ear canal with the hearing aid in the ear.

- Select **Quick Fit** to make global fitting adjustments. Select **Fine Tuning** on the bottom of the Quick Fit screen or from the **Flyout Menu** ≡ to make detailed fitting adjustments of individual channels and input levels.

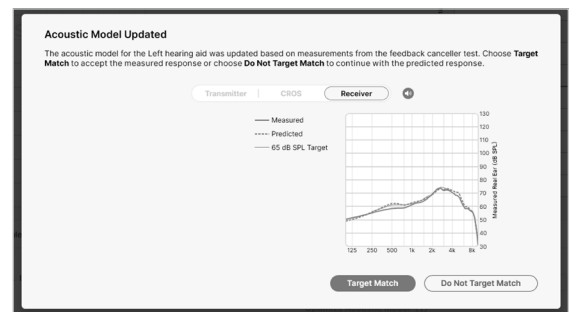


Figure 7a

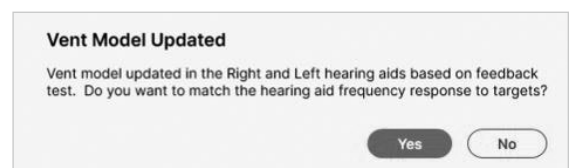




Figure 7b

Fitting (Continued)

- 4 Select **CROS/BiCROS** from the **Flyout Menu** . Select the **Balance Control** pin to adjust the level of sound coming from the Transmitter relative to the sound coming from the receiver. [Fig. 8]

NOTE: BiCROS Balance Control settings apply to all BiCROS programs. Each step results in a change of +1 dB on one side and -1 dB on the other side, resulting in a 2 dB step size. The maximum difference between the Transmitter and the Receiver is 10 dB.

- 5 Select **Sound Manager** to make adjustments to environmental sound management features and directionality.
- 6 If needed, select **User Controls**. Drag any user control you wish to enable from the left and drop it on a gesture on the right. [Fig. 4]
- 7 Select **Indicators**. Click the check boxes to enable or disable indicators. [Fig. 9]

- **CROS Stream Start:** This indicator plays when the user navigates to a program where CROS/BiCROS is enabled from a Hearing Aid Only program. Also, this indicator will play when the Transmitter is powered on after the Receiver is powered on and in a CROS/BiCROS program.
 - **CROS Stream Stop:** This indicator plays when the Transmitter signal stops. This could be due to the Transmitter being turned off, moving out of range or having a dead battery.
 - **Balance:** This indicator plays when adjustments are made to the Balance user control. Individual indicators for **Maximum, Default, Minimum, Home** and **Steps** can be enabled or disabled. The Maximum balance indicator will play when the balance has been shifted to maximize sound coming from the Transmitter. The Minimum balance indicator will play when the balance has been shifted to minimize sound coming from the Transmitter.
- 8 Adjust **All Indicators** or individual indicators to increase or decrease volume level for all Tone or Speech indicators in 5 dB increments.
 - 9 The user will hear a special tone after the Low Battery and Mute indicators for the Transmitter. This helps the user determine which side has a low battery or is being muted. To demonstrate the indicators for each side, select **Receiver or Transmitter**, then select either the Hearing Aid icon or the Computer icon.
 - 10 Utilize the **Flyout Menu**  and the **Left Menu Bar** to access additional screens and help features. [Fig. 5]

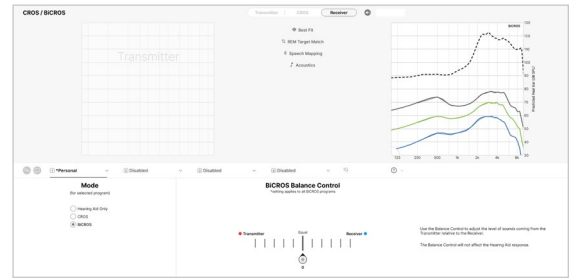


Figure 8

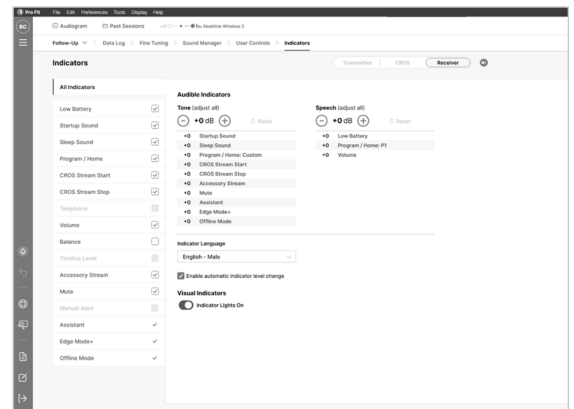


Figure 9

Follow-Up

Following the Fitting session, Pro Fit will advance to the Follow-Up session automatically. [Fig. 10]


- 1 Review **Data Log**, if desired.
- 2 If needed, select **Fine Tuning** to make detailed adjustments of individual channels and input levels.
- 3 Select **Sound Manager** to make adjustments to environmental sound management features and directionality.
- 4 If needed, select **Indicators** to adjust the volume level and stimulus type for indicators. [Fig. 9]
- 5 If needed, select **User Controls**. Drag any user control you wish to enable from the left and drop it on a gesture on the right. [Fig. 4]
- 6 Utilize the **Flyout Menu**  and the **Left Menu Bar** to access additional screens and help features. [Fig. 5]



Figure 10

Audio Streaming Demo

Audio Streaming* is available on the **Flyout Menu** ≡ and allows you to demonstrate streaming audio from a paired phone or accessory and adjust the frequency response with Quick Fit or Fine Tuning controls. [Fig. 11]

- 1 Select Audio Streaming from the **Flyout Menu** ≡ [Fig. 5].
- 2 Start streaming from the paired phone or accessory (Bluetooth® must be turned on).
- 3 Select **Start Demo Mode**. Once the stream is detected, “Now Streaming” displays [Fig. 11].
- 4 Select **Mute stream** (talk to patient) if desired, then select **Unmute** stream to start the stream again.

NOTE: If streaming from an Android phone or a device using the ASHA streaming protocol, the stream cannot be muted via Pro Fit. Select **Stop Demo Mode** to talk to the patient.

- 5 On the Audio Streaming screen, select **Quick Fit** to make global fitting adjustments. Select **Fine Tuning** to make detailed fitting adjustments of individual channels and input levels.
- 6 Select **Stop Demo Mode** to stop the demo/stream.
- 7 To recalculate the frequency response to match the prescriptive target, select **Stream Target Match**.

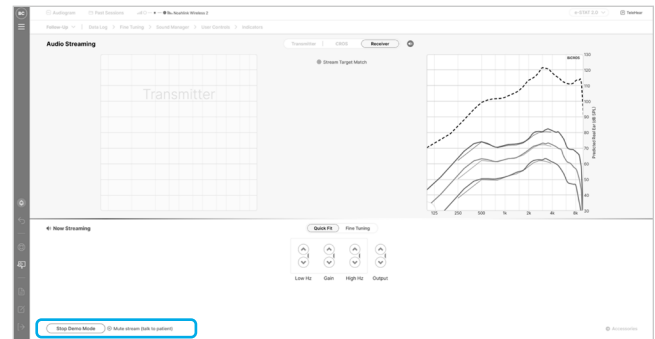



Figure 11

Session Notes

Session Notes* tracks and saves adjustment history and allows you to enter detailed fitting comments that are saved with each session. Session Notes can be viewed for reference, ensuring continuity of care and facilitating future follow-ups or troubleshooting.

- 1 Select **Session Notes**  from the **Left Menu Bar**. [Fig. 5]
- 2 Enter individual topics that required fitting adjustments and select **Add** for each one. Each topic will display under **Today's Topics**. [Fig. 12]
- 3 Fitting adjustments will be tracked under **Today's Adjustments**. Drag and drop each adjustment to the corresponding **Today's Topic**. Select **Check at next session** or **Topic resolved**. [Fig. 12]
- 4 Use **Additional Notes** to add free-form notes.
- 5 Exit Pro Fit and enter a session title to save. Select **Yes**.
- 6 Saved fitting sessions with **Session Notes** will have an eye icon next to it on the Noah home screen. Click the eye icon to quickly display the **Report Viewer**.

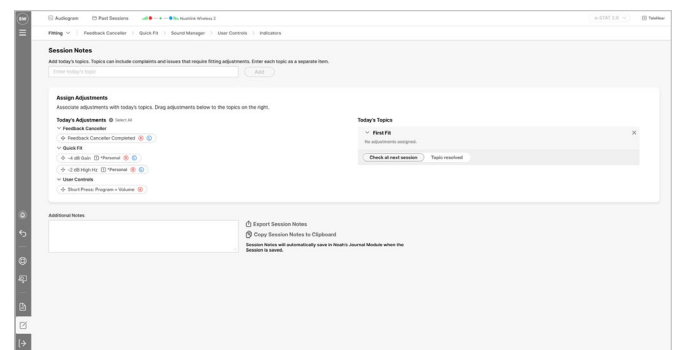


Figure 12

NOTE: If you select **Check at next session** the topic will display under **Session Notes** as **Last Visit's Topic** and you will be asked if the topic was resolved.

CROS/BiCROS Mobile App Control

The Balance screen will be visible in the mobile app if at least one BiCROS memory has been programmed in Pro Fit.

To adjust the Balance Control:

- 1 Tap **My Hearing** in the mobile app, then tap **CROS Balance**. [Fig. 13]
- 2 Drag the circle icon along the slider towards the **Receiver** for more sound from the Receiver and less sound from the Transmitter. Drag the icon towards the **Transmitter** for more sound from the Transmitter and less sound from the Receiver. [Fig. 14]

To Mute/Unmute the Transmitter:

- 1 Tap the **Mute transmitter** button to mute the transmitter. Tap again to unmute. [Fig. 14]


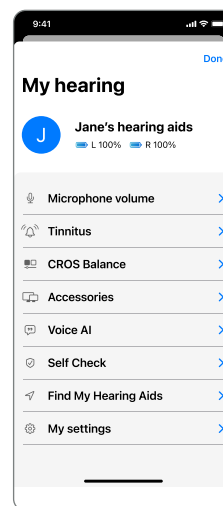


Figure 13

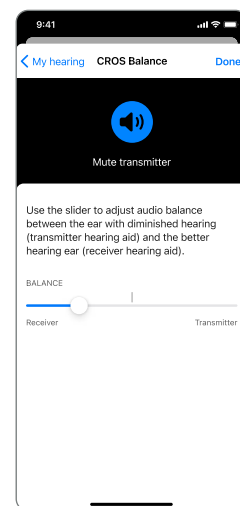

Unpairing

To unpair a CROS System and convert the Receiver back to a traditional hearing aid, you must return the hearing aid to manufacturer settings:

- 1 On the Connect screen, select the **Receiver** only.
- 2 Click **Tools** from the menu bar.
- 3 Select **Clear Settings and Reset Device**. This will clear all patient data and settings, including all program-specific CROS/BiCROS information.
- 4 When the process is complete, select **OK**.

To learn more, visit www.starkeypro.com/products/other-products/software/pro-fit
OR www.audibelpro.com/products/other-products/software/pro-fit

Pro Fit, StarLink and Starkey logo are trademarks of Starkey Laboratories, Inc. Android is a trademark of Google, LLC. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. Any use of such marks by Starkey is under license.
©2025 Starkey Laboratories, Inc. All Rights Reserved. 8/25 FLYR4219-03-EN-SC

