

QUICKTIP

OrCam Vision Aid Compatibility



OrCam® MyEye is a device that is used by visually-impaired persons to help them interact more effectively with the world. OrCam can recognize and read text, identify currency, learn and recognize faces, identify products and more. When paired to a compatible hearing aid,* the audio from the OrCam MyEye is streamed wirelessly to the hearing aid.

Pairing

OrCam MyEye will pair to one hearing aid. It is recommended to pair to the hearing aid on the same side of the head as the OrCam device for more reliable wireless transmission.

- 1 Power on the OrCam device by pressing and holding the power button for two seconds. After about a minute, you will hear "OrCam is ready. Battery is X% charged."
- 2 Navigate to **Main Menu**.
- 3 Following the commands from OrCam, navigate to: **Connectivity Settings Menu > Bluetooth® Settings Menu > Connect to a Bluetooth Audio Device**. This will put OrCam into discovery mode.
- 4 Power off, then power on the hearing aid to place in pairing mode.
- 5 OrCam will announce the name of the hearing aid and serial number. Touch the OrCam **Touch Bar** to pair.

Use

- Stream volume is controlled using the touch bar on the OrCam MyEye. Swipe forward to increase volume and backwards to decrease volume. Stream volume for OrCam cannot be controlled via the hearing aid's user controls or hearing aid mobile app.
- If while streaming from the OrCam MyEye the hearing aid wearer receives a phone call, the OrCam stream will be stopped automatically and the phone call will be prioritized.
- If streaming from the OrCam MyEye and the hearing aid wearer wants to start a stream from a 2.4 GHz accessory, the OrCam stream will be stopped automatically and the accessory stream will be prioritized.
- If streaming from a 2.4 GHz accessory and the hearing aid wearer wants to start streaming from the OrCam MyEye, the accessory stream must be stopped manually before streaming can begin from the OrCam device.



To purchase an OrCam device and/or for OrCam information, instructional videos and FAQs, please visit www.orcam.com.

*OrCam MyEye is compatible with most AI hearing aids; please contact your rep for specific information.

Images not shown to scale. | The Starkey logo and Thrive are trademarks of Starkey Laboratories, Inc. | OrCam is a trademark of OrCam Technologies, LTD. | The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Starkey is under license. | ©2021 Starkey Laboratories, Inc. All Rights Reserved. 1/23 FLYR3653-03-EN-SC

