



MED⁹EL

DualSync Technology

Experience the best in
bimodal hearing





“I am very satisfied with my overall listening experience. The improved clarity is incredible. The sound is so crisp and clear you almost will have to turn the volume down.

You will appreciate the ease and quality of DualSync immediately.”

Jen Murray
Retired Interior Designer

A purposeful pair

Everyone deserves to hear their best—and now there is a breakthrough streaming solution you can offer to bimodal patients.

Starkey is excited to partner with MED-EL, a pioneer in hearing implants, to offer bimodal streaming for patients who use both a cochlear implant and a hearing aid. Called DualSync Technology, it enables Apple® devices to recognize the audio processor and hearing aid as a pair for a streamlined bimodal streaming experience.



A better listening experience

DualSync Technology is designed to provide your bimodal patients wearing Starkey and MED-EL technology with numerous benefits, including:

- Improved synchronization
- Better spatial awareness localization
- The ability to stream calls and media from iPhones®
- Less echo
- And more



Partners in transformative technology

As leaders in their respective industries, Starkey and MED-EL are ideal partners for your practice.

Starkey technology, in your hands, gives patients the opportunity to hear and live their best lives possible.

MED-EL, the world's leading hearing implant company, is on a mission to help people with hearing loss experience the joy of sound through innovation.

Starkey Stock and Custom Cased Receivers

SnapFit 2.0 Stock and Custom Cased Receivers



Comfort Bud Open
5 mm, 6 mm, 7 mm, 9 mm, 11 mm

Comfort Bud Occluded
4 mm, 5 mm, 6 mm, 7 mm, 9 mm, 11 mm

Power Dome
8-10 mm, 10-12 mm, 12-14 mm

Custom Cased Receiver
Custom shell with embedded receiver


Stock Silicone Helix RIC Earmold

Stock Hollow Silicone RIC Earmold

Stock Silicone RIC Earmold

Fitting Ranges

Visit StarkeyPro.com/OmegaAI for full technical data sheets.

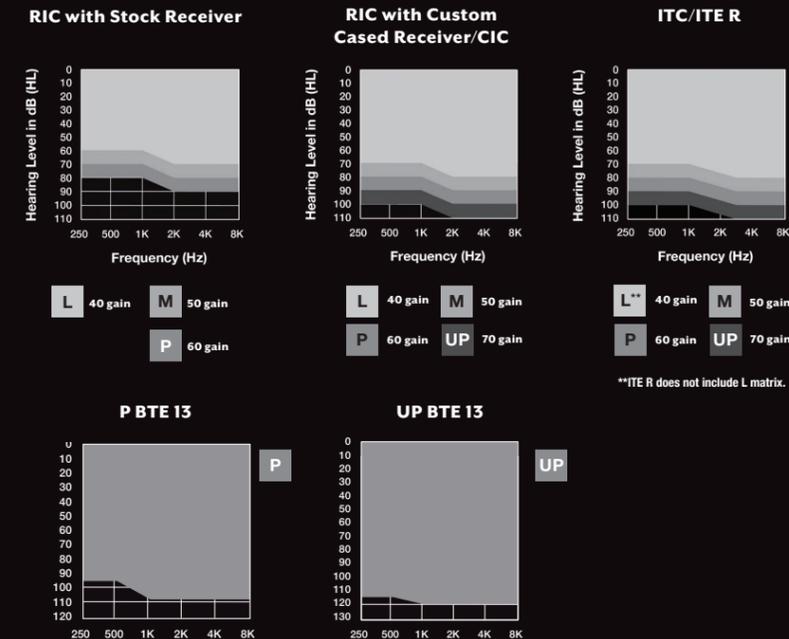

Omega AI
Starkey's newest hearing aids help patients hear life in all its dimensions.



Edge AI and G Series AI
From custom to BTE, Starkey's product families offer options for all lifestyle and hearing needs.



My Starkey
Starkey's companion app helps patients enjoy greater listening moments, improved well-being, and easier everyday living.

Starkey Omega AI, G Series AI, Edge AI, and Genesis AI wireless devices of any style are compatible with MED-EL SONNET 3, SONNET 2, SONNET or RONDO 3 processors. SONNET 2, SONNET, and RONDO 3 require AudioStream.



SONNET 3
MED-EL's latest technology, their smallest and lightest behind-the-ear audio processor with integrated direct streaming, is ready to connect patients with what they care about most.



RONDO 3, SONNET 2 and SONNET
From their incredible comfort to complete convenience, these audio processors make all-day hearing simple.



AudioKey App
Lets patients control audio processor settings, check on their child's hearing, view detailed stats, and locate lost audio processors—without carrying a remote.




*If SONNET 3 or AudioStream is paired to the iPhone, forget it via the Accessibility menu. If My Starkey app is installed, completely close or uninstall it before bimodal configuration. [The app may be reopened or installed after configuration is complete.]

Bimodal Streaming

To set up bimodal streaming on your patient's iPhone, follow these easy steps:

- 1** Pair the hearing aid from the **Accessibility** menu under **Settings**.*
- 2** Open the **AudioKey** app, select **connectivity options**, and then follow the **configuration steps** in the app.
- 3** Stream favorite audio to both devices simultaneously.

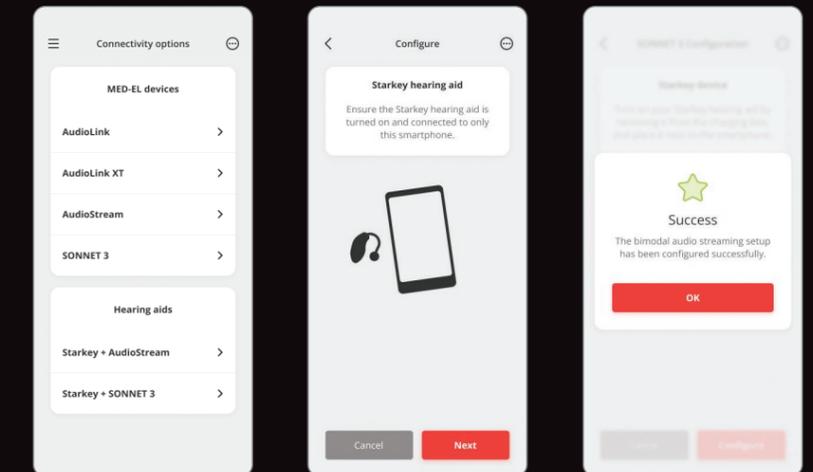

To learn more about DualSync Technology,
reach out to your Starkey representative.



Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries and regions. App Store is a service mark of Apple Inc. | Android is a trademark of Google LLC. Google Play and the Google Play logo are trademarks of Google LLC. Sonnet, Rondo, AudiKey, MED-EL and MED-EL logo are trademarks of MED-EL Elektromedizinische Geräte Ges.m.b.H. Starkey logo, Starkey, Starkey Edge and Genesis are registered trademarks of Starkey Laboratories, Inc. Omega, DualSync, Starkey G Series are trademarks of Starkey Laboratories, Inc.

©2026 Starkey Laboratories, Inc. All Rights Reserved. 1/26 BKLT3285-00-EN-ST