

QUICKTIP

Premium Standard Charger & Premium Custom Charger



The Premium Standard Charger operates as a portable charging station, dry-aid kit and case for standard lithium-ion rechargeable hearing aids, including RIC R and BTE R. The Premium Custom Charger operates as a portable charging station and case for custom lithium-ion rechargeable hearing aids. Both chargers are equipped with LED lights to indicate charging status for both the hearing aids and the on-board battery.

LED Indicators for Hearing Aid Charging

The front of the charger is equipped with two LED indicators to provide information about hearing aid charging.

- When the hearing aids are placed into the charger, the front LED lights will glow green to indicate successful charging.
- When the hearing aids are fully charged, the front LED lights will be solid green indicating a full charge.
- If there is an issue with charging and the hearing aids are in a fault state, the front LED lights will blink red.

NOTE: If a fault state is detected, remove the hearing aids from the charger, wait until the LEDs turn off, and re-insert the hearing aids. If the fault state continues, contact your hearing professional.



LED Indicators for On-board Battery

The back of the charger is equipped with four LED indicators to provide information about the status of the on-board battery. The on-board battery allows charging of the hearing aids to occur while the charger is away from a power source.

- Four solid green lights indicate the charger is plugged into a power source and is fully charged.
NOTE: *When charging without the power cord, the back LEDs will turn off after 10 seconds to preserve battery life.*
- When away from a power source, four solid lights indicate the battery has more than 75% charge.
- Three solid lights indicate the battery has less than 75% charge.
- Two solid lights indicate the battery has less than 50% charge.
- One blinking light indicates that the charge is low, and the charger should be connected to a power source to charge the on-board battery.

NOTE: *When plugged into a power source, both the hearing aids and on-board battery will charge simultaneously.*



Use

- 1 The on-board battery in both the Premium Standard Charger and Premium Custom Charger will be fully charged in 2.5 hours. Standard and custom hearing aids will be fully charged in 3.5 hours.
- 2 Hearing aids can be charged up to 3 times via the on-board battery without being connected to a power source.
- 3 It is safe to keep the hearing aids in the charger as they cannot overcharge.
- 4 Charging will occur with the lid open or closed.
NOTE: *It is recommended to keep the Standard Charger lid closed to preserve the desiccant puck if also using as a dehumidifier.*
- 5 Hearing aids automatically power on when removed from the charger and power off when inserted into the charger.
- 6 If the charger is not plugged in and runs out of charge, the hearing aids will turn ON in the charging case.