

Auracast Assistant

Auracast™ makes it easier to hear in shared spaces by broadcasting audio directly to nearby hearing aids. Using Auracast assistant, you can quickly select the stream you want in your hearing aid app and start listening.



Watch the video
to learn more!

Connecting to an Auracast Broadcast

- 1 Within the hearing aid mobile app, tap the **Stream options** (🔊) icon.
- 2 Tap **Auracast broadcasts** to start a search for available broadcasts.
- 3 Tap the broadcast you would like to join. If a broadcast is password protected, a pop-up will prompt for a password.
NOTE: Not all Auracast broadcasts will support hearing aid streaming. Available broadcasts will be separated into two categories, supported and unsupported. Supported broadcasts are those that support hearing aid streaming. Broadcasts that do not support hearing aid streaming will be listed in the Unsupported section.
- 4 Once connected, the current broadcast will appear at the top of the screen.
- 5 If the broadcast is available in more than one language, an option will appear to allow for language selection.
- 6 To momentarily stop and restart the audio stream, tap the **Stop** or **Play** icon.
- 7 From the Home screen, the Stream options button will update to show status for any stream. The icon will turn gray when there is no active audio stream. It will turn blue and animate when audio is streaming.
- 8 To disconnect from the broadcast, select the Auracast stream name to navigate back to the Streaming options screen. Tap **Disconnect broadcast**. The broadcast will automatically be disconnected when you walk out of range.

