

QUICKTIP

My Starkey

Android Version



Downloading My Starkey

You must have a Google Play Store account to download My Starkey. A Google Play Store account can be created by going to **Settings>Accounts**. Choose an existing account or select **Add Account** to create a new account.

- 1 Once an account is set up, open the Play Store.
- 2 Search for **My Starkey**.
- 3 Select **Install**.



My Starkey

My Starkey Pairing and Initial Setup

My Starkey will search for and pair to the hearing aids. Pairing should be completed after the hearing aids have been programmed in the Pro Fit software. Ensure that Pro Fit is closed prior to pairing with any Android device.

- 1 Tap on **Get Started** to begin the app set-up process.
- 2 Review the Terms of use. Tap **Accept** to continue.
- 3 Sign in, create an account with an e-mail and password, or select **Skip for now** to postpone account creation. Cloud connectivity allows for TeleHear, Fall Alert and for app/hearing aid information to be backed up and restored.
- 4 Users are asked to provide consent for My Starkey to collect and store Hearing aid data, Activity tracking, and Analytics. Tap the slider to disable a consent, if desired. Select **Continue**.
- 5 Select **Continue** to allow My Starkey access to the user's location and to share notifications.
- 6 Users are asked to allow My Starkey to send the user notifications. Select **Allow** or **Don't Allow**.
- 7 Ensure the hearing aids are charged and turned on. Select **Continue**.
- 8 A prompt will inform the user that Nearby Devices Permission is required. Tap **OK**, then select **Allow**.
- 9 When prompted, select **Pair**, then tap **Agree** to allow My Starkey to access the hearing aids.
- 10 My Starkey will connect to the hearing aids. Select **Continue**.

You're All Set. Select **Got it** to enter My Starkey.

Features may vary by country.

The app may look slightly different based on your phone and/or hearing aid model.
Android and Play Store are trademarks of Google, LLC.
Starkey, Starkey logo, Pro Fit, Edge Mode and TeleHear are trademarks of Starkey Laboratories, Inc.
©2024 Starkey Laboratories, Inc. All Rights Reserved. 3/25 FLYR4084-03-EN-ST

