


## QUICKTIP

# Auracast Assistant

Auracast™ is a feature of Bluetooth® LE Audio that improves accessibility by allowing a single audio source broadcast sound to an unlimited number of nearby devices, offering high-quality, low-latency audio. Auracast assistant\* allows the hearing aid user to select the Auracast stream they want to listen to using the hearing aid mobile app to easily connect to an Auracast broadcast.

## Connecting to an Auracast Broadcast

- 1 Within the hearing aid mobile app, tap the **Stream options** (🔊) icon.
- 2 Tap **Auracast broadcasts** to start a search for available broadcasts.
- 3 Tap the broadcast you would like to join. If a broadcast is password protected, a pop-up will prompt for a password.  
**NOTE:** *Not all Auracast broadcasts will support hearing aid streaming. Available broadcasts will be separated into two categories, supported and unsupported. Supported broadcasts are those that support hearing aid streaming. Broadcasts that do not support hearing aid streaming will be listed in the Unsupported section.*
- 4 Once connected, the current broadcast will appear at the top of the screen.
- 5 If the broadcast is available in more than one language, an option will appear to allow for language selection.
- 6 To momentarily stop and restart the audio stream, tap the **Stop** or **Play** icon.
- 7 To disconnect from the broadcast, tap **Disconnect broadcast**. The broadcast will automatically be disconnected when you walk out of range.
- 8 From the Home screen, the Stream options button will update to show status for any stream. The icon will turn gray when there is no active audio stream. It will turn blue and animate when audio is streaming.

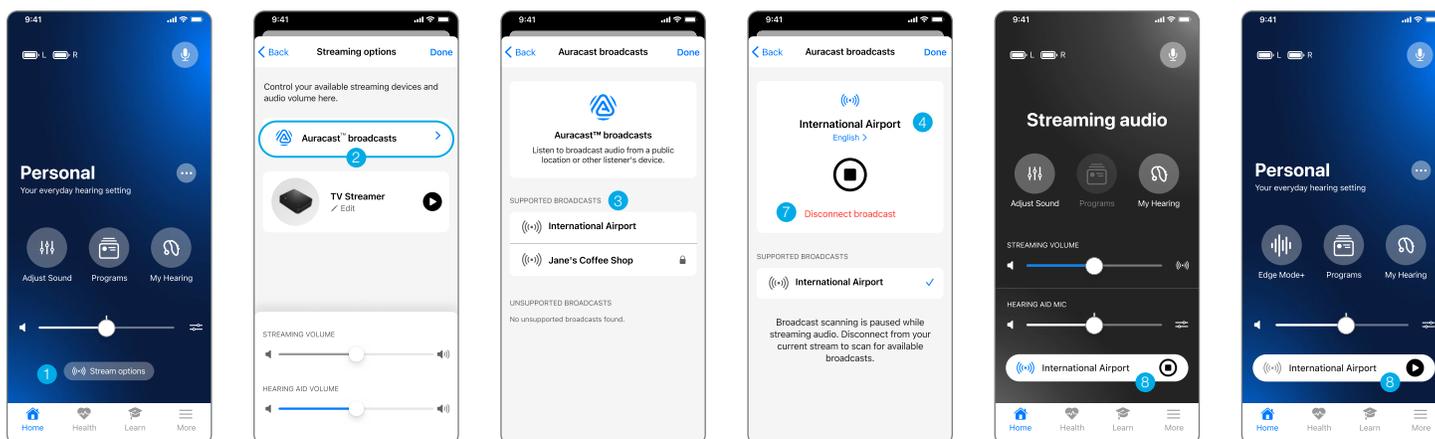

\*Available on select hearing aids. Contact your rep for specific information.  
The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc.  
The Auracast™ word mark and logos are trademarks owned by the Bluetooth SIG. Any use of such marks by Starkey is under license.  
Neuro Processor is a trademark of Starkey Laboratories, Inc. The Starkey logo is a registered trademark of Starkey Laboratories, Inc.  
©2026 Starkey Laboratories, Inc. All Rights Reserved. 01/26 FLYR4546-00-EN-ST

