

QUICKTIP

Preparing for a TeleHear Appointment with your Patient using Inspire X and Thrive App

The TeleHear remote programming feature allows you to make remote programming adjustments in real-time using a two-way audiovisual link between you and your patient.

Set up your Office

You will need:

- Fitting computer
- Inspire X 2022.1 or greater
- Internet connection
- Internet browser (Google Chrome or Microsoft Edge)
- Patient database (NOAH or PatientBase)
- Integrated computer camera or external camera (webcam)
- Integrated computer microphone/speakers or headset

**Getting Started:**

- **Sign in to TeleHear through Inspire X.**
- **Generate a remote programming invite code in Inspire X.** The remote programming invite code creates a secure cloud connection between you and your patient. This code is generated once per patient and is only used during setup. A code can be generated with or without the hearing aids connected to the software. Codes are valid for three days and each patient must receive a unique code. Make note of the remote programming code for later use.

[Government Services TeleHear System Setup with Inspire X and Thrive App QuickTip](#)

Tips for a successful remote programming session

- Position your camera to reduce visual distractions such as windows or high traffic areas.
- Select a quiet location for minimal auditory distractions.
- Check that your camera, microphone, and speakers are working properly.
- Position yourself close to the microphone and speak clearly for the best sound quality.
- Position yourself so that your patient can clearly see your face.
- Avoid unnecessary noises (i.e., papers rustling, clicking a pen) as these sounds can negatively affect sound quality during the live session.
- Use your clinical judgement to determine if the patient needs to come into the office for further assistance.

Prepare your Patient

They will need:

- Thrive platform 2.4 GHz hearing aids
- Smart device
- Thrive Hearing Control app
- Internet connection (cellular or wifi)

Getting Started:

The following tasks will ensure that your patient is ready to use TeleHear.

- **Download the Thrive Hearing Control app.** The Thrive Hearing Control app is available for both Apple® and Android™ smart devices. For compatibility information visit: <https://www.starkey.com/hearing-aids/apps/thrive-hearing-control/compatibility>.

- **Pair the hearing aids to the smart device.**

For assistance, refer your patient to the Thrive Setup QuickTIPS.

- **Complete the Thrive app setup process, including cloud sign-in.**


The patient must be signed into the cloud to use TeleHear.

For assistance, refer your patient to the Thrive Setup QuickTIPS.

- **Provide the remote programming invite code that was generated in Inspire X to your patient, and have them enter it into the Thrive app.** This creates a secure cloud connection between you and your patient.

For step-by-step instructions refer your patient to TeleHear Appt Prep for Patients with Thrive App QuickTIP.

- **Schedule a time for the first TeleHear appointment.**

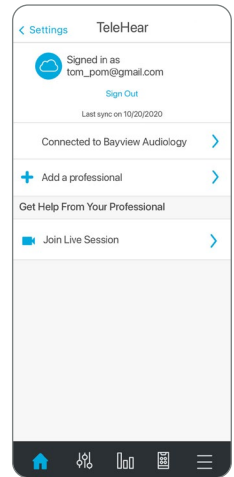

Instructional resources to share with your patient:

- [TeleHear Appt Prep for Patients with Thrive App QuickTIP](#)
- [TeleHear-Join Live Session](#)
- Patient Product Support: www.starkey.com
- Veteran Technical Support Line: 1-800-211-7789