


Edge Mode+



The industry's first-of-its-kind intelligent, on-demand noise management and comfort feature — Edge Mode+ scans, adapts, and optimizes as the wearer changes environments or as the environment they're in changes.

Now available on all Starkey Edge AI and Genesis AI wireless styles, as well as Starkey Signature Series CIC R NW.

Edge Mode+ has been shown to improve speech recognition and decrease listening effort.¹

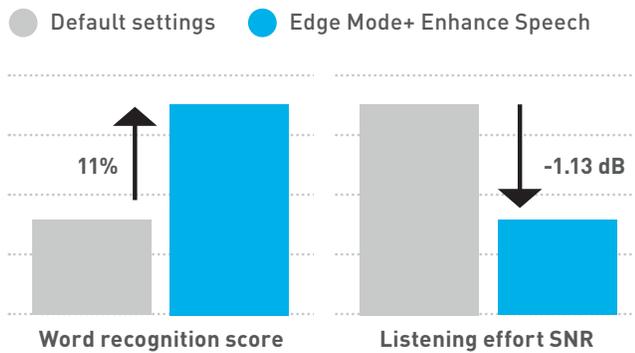

EDGE^{AI} *Signature* SERIES
GENESIS^{AI}



Where to find it

- My Starkey mobile app *
- User Control
- Remote Control*
- Automatic*

*Edge AI and Genesis AI styles only.

Edge Mode+



Activate Edge Mode+ then select your listening intent for further personalization.



Reactivate Edge Mode+ to rescan as environment changes.

Automatic Edge Mode+



Activate Edge Mode+ then select your listening intent for further personalization.



Automatically rescans as environment changes.


Edge Mode+ Set-up & Use Instructions

Edge Mode+ is now available on all Starkey Edge AI and Genesis AI wireless styles, as well as Starkey Signature Series CIC R NW. It is defaulted as an option on the homescreen of the My Starkey app, but may also be configured in the hearing aid User Controls and/or the Remote Control 2.0 Favorite Button. See below for details on engaging Edge Mode+.

Hearing aid user control

In Pro Fit select **User Controls** on the Flyout Menu ☰.


Drag **Edge Mode+** control from the left and drop to either **Short Press** or **Double Tap** gesture on the right.

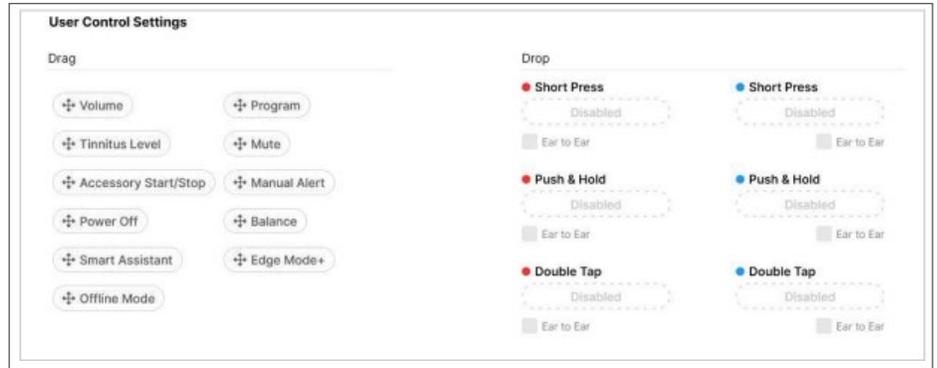

Remote control* using the ☆ Favorite Button

In Pro Fit select **Accessories** on the Flyout Menu ☰.


Use the dropdown menu below the Remote image to set the Favorite button configuration.

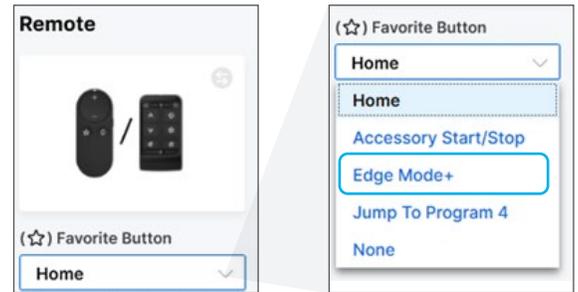

My Starkey mobile app*

1. Open the My Starkey mobile application.
2. Tap the **Edge Mode+** icon on the home screen.
3. Best Sound is automatically applied; for further personalization based on the patient's listening intent: speech or reduced noise, the patient can select either Enhanced Speech or Reduce Noise.
4. Tap **Save**.
5. To exit Edge Mode+, select **Programs** and select the desired program.

*Edge AI and Genesis AI styles only.

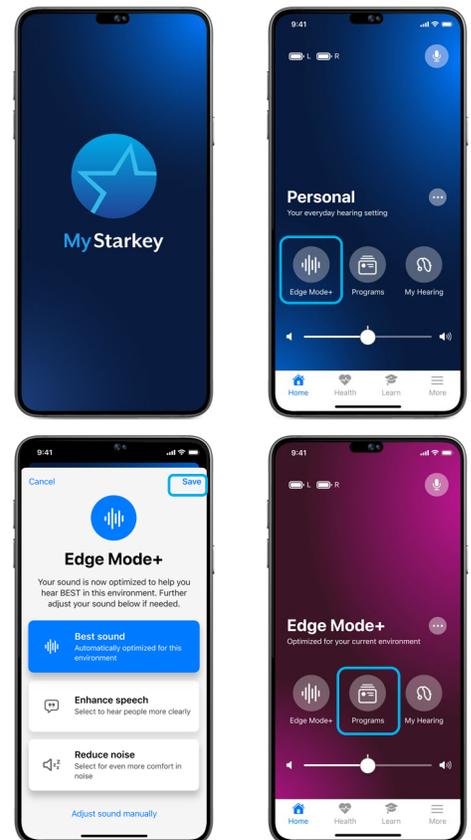

¹Jaekel, B., & Xu, J. (2024). Edge Mode+: On-demand processing improves speech recognition and listening effort in hearing-aid users. *Audiology Practices*, 15(4), 8-16. Read more about Starkey research here: <https://www.starkeypro.com/continue-learning/research-and-publications>. ©2024 Starkey Laboratories, Inc. All Rights Reserved. 8/24 FLYR4146-02-EN-GS P00006162 Starkey logo, Genesis, Pro Fit and Edge Mode are registered trademarks of Starkey Laboratories, Inc. Starkey Edge and Starkey Signature Series are trademarks of Starkey Laboratories, Inc.