(A) audibel

Eb







Eclipse II

table of contents FDA information

your hearing instruments	1
CIC	
your CIC instruments	2
identification	3
batteries	4
insertion and removal	6
on, off and volume	8
multimemory settings	9
using the telephone	10
feedback	11
ITC	
your ITC instruments	12
identification	13
batteries	14
insertion and removal	16
on, off and volume	18
multimemory settings	20
directional microphones	21
using the telephone	22
feedback	24
ITE	
your ITE instruments	25
identification	26
batteries	27
insertion and removal	29
on, off and volume	31
multimemory settings	33
directional microphones	34
using the telephone	35

feedback

37

battery warning	38
learning to hear well again	39
realistic expectations	40
the communication process	41
handling and wearing	42
adjusting to a noisy world	43
start in quiet, familiar	
places	44
focusing on specific	
sounds	45
hearing the TV and radio	46
enjoy, but don't overdo it	47
how family & friends	
can help	48
speak clearly and normally	49
minimize distractions	50
time for added	
consideration	51
let others know what	
you need	52
hearing instrument care	53
putting it in the right place	54
daily cleaning and care	55
avoiding moisture	56
service and repair	57
troubleshooting guide	58
FDA information	60

that all medically treatable conditions that may affect hearing are identified and treated before the hearing aid is purchased.

Following the medical evaluation, the physician will give you a written statement that states that your hearing loss has been medically evaluated and that you may be considered a candidate for a hearing aid. The physician will refer you to an audiologist or hearing aid dispenser, as appropriate, for a hearing aid evaluation.

The audiologist or hearing aid dispenser will conduct a hearing aid evaluation to assess your ability to hear with and without a hearing aid. The hearing aid evaluation will enable the audiologist or dispenser to select and fit a hearing aid to your individual needs.

If you have reservations about your ability to adapt to amplification, you should inquire about the availability of a trial-rental or purchase-option program. Many hearing aid dispensers now offer programs that permit you to wear a hearing aid for a period of time for a nominal fee after which you may decide if you want to purchase the hearing aid.

Federal law restricts the sale of hearing aids to those individuals who have obtained a medical evaluation from a licensed physician. Federal law permits a fully informed adult to sign a waiver statement declining the medical evaluation for religious or personal beliefs that preclude consultation with a physician. The exercise of such a waiver is not in your best health interest and its use is strongly discouraged.

CHILDREN WITH HEARING LOSS. In addition to seeing a physician for a medical evaluation, a child with a hearing loss should be directed to an audiologist for evaluation and rehabilitation since hearing loss may cause problems in language development and the educational and social growth of a child. An audiologist is qualified by training and experience to assist in the evaluation and rehabilitation of a child with a hearing loss.

©Audibel Electronics 6700 Washington Avenue South Eden Prairie, MN 55344