

D [REDACTED]
[REDACTED]
[REDACTED]

E [REDACTED]
[REDACTED]








F [REDACTED]
[REDACTED]

G [REDACTED] ULOJPOUWBCP OUVXBX

H 1PX.BEINJFOJBJ[REDACTED]NOJFOJB%PULOJPOUWBCP
LPOUOVXBX

I [REDACTED]SJWF
[REDACTED]PTU'RP
[REDACTED]LV
[REDACTED]

J [REDACTED]OUOVXBX

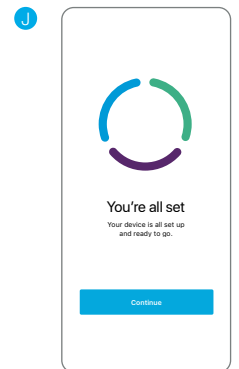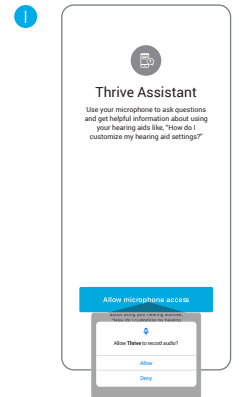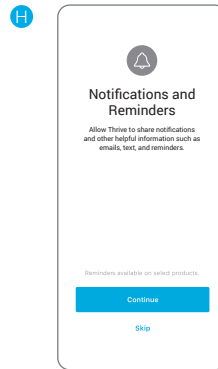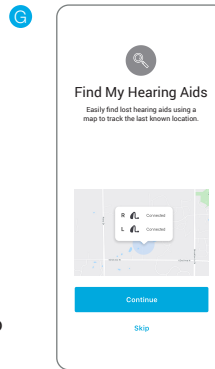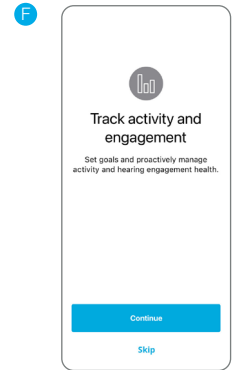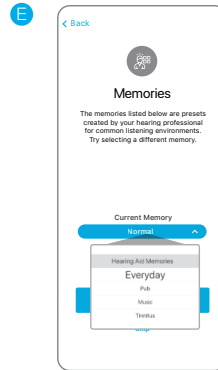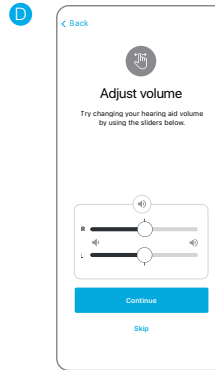



Device Settings

Advanced settings for the hearing aids can be turned on/off or set in this menu.

About My Devices: Displays the model, technology level and serial numbers of the hearing aids as well as the current firmware version.



Alerts: Turning on Alerts allows an indicator to be played in the hearing aids when receiving a phone call. Press the blue button to preview the alerts.

Tap Gesture: Turn the tap gesture on or off. Double Tap feature controls Accessory Start/ Stop Stream, Edge Mode and Thrive Assistant. Set preferred functionality for the right and left hearing aids by tapping the pencil icon . Set the tap sensitivity to **Less**, **Default** or **More**. Available for AI hearing aids only.

Car: Turn Automatic Car on or off. Tap **OK** to allow access to the phone's motion sensor to provide automatic memory changes when in a vehicle achieving a speed of 10 miles per hour or greater. Set which memory the Automatic Car mode uses by clicking the  and selecting the preferred memory.


Data Log: Turn Data Log on or off. Turning Data Log off eliminates the ability to generate an Engagement Score.

Auto Streaming: Turn Auto Streaming on or off. Auto Streaming is on by default following programming in Inspire X. Auto Streaming will use Stream Boost by default. If Auto Streaming is turned off, the memory will not change when streaming.

- **Phone Call:** Select which memory the Automatic Phone mode uses by clicking on the  and selecting the preferred memory. Also select how the hearing aid microphones will perform during phone call streaming. Tap **No Change**, **Decrease** (50%) or **Mute** to indicate microphone preference.
- **Media Audio:** Select which memory the automatic media streaming mode uses by clicking on the  and selecting the preferred memory. Also select how the hearing aid microphones will perform during media streaming. Tap **No Change**, **Decrease** (50%) or **Mute** to indicate microphone preference.

Auto On/Off: Tap the slider to deactivate/activate Auto On/Off. Auto On/Off allows the hearing aids to automatically go into low power mode when not in use. Available for AI hearing aids only.

Thrive Assistant Mic: Tap **HA Mic** or **Phone Mic** to select preference for which mic is used by Thrive Assistant. HA Mic is the default setting.

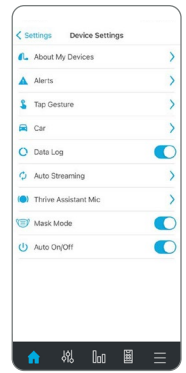

Wellness Score Setup

Set activity tracking information to ensure the most accurate calculations of the Wellness Score.

- Activity Tracking: Customize activity score goals including Steps, Exercise and Stand. Information can be modified at any time by going to the **Settings > Wellness Score > Activity Tracking**.

NOTE: Direct streaming compatible with certain Android devices only. For all other Android smartphone models, hearing aids and the 2.4 GHz Remote Microphone + work together to deliver hands-free phone conversations with binaural streaming. Audio such as music, movies, GPS directions, and more can also stream through the Remote Microphone + directly to the hearing aids. For more information, see the Remote Microphone + QuickTIP.