



Starkey | Technical Data

CIC

Completely-In-Canal

Omega AI

Key Features

- Acoustic Optimization Model 2.0
- Auto Stream
- Find My Hearing Aids
- Bluetooth® Low Energy Audio with *Auracast™* technology
- StarLink Edge Remote Microphone, StarLink Edge Table Microphone, StarLink Edge TV Streamer & StarLink Remote Control 2.0 Compatibility
- Apple Watch® Controls*
- Balance Assessment within My Starkey App
- Demo Stream
- Tinnitus maskers (blue/red/pink/purple/ocean)
- Zinc Air Battery
- 24 Channels
- 4 Programs**
- Additive Compressor
- Feedback Cancellor
- Data Log
- Voice AI*
- Translate
- Multiflex Tinnitus Pro
- Transcribe
- Edge Mode+
- Gen AI Smart Assistant & TeleHear AI
- 2-Way Audio & Find My Phone
- Hearing Health & Activity Tracking
- Fall Alert
- Reminders & Self Check
- Auto Sleep
- Tap Control
- TeleHear full remote fit & REM Target Match
- My Starkey & Hear Share App Compatibility
- NOAH Link Wireless Programming
- Wireless Programmability

Sound Manager:

- Quiet
- Machine Noise Reduction
- Speech in Noise Reduction
- Transient Noise Reduction
- Wind Reduction

Ear-to-Ear Technology:

- Binaural Wind
- Binaural Machine

Multiple Processor:

For many years, Starkey has utilized multiple processors within a single chip, processors that work separately and concurrently to perform the various processing tasks of a modern-day hearing aid. Each processor is optimized to provide efficient calculations for a particular aspect of the audio processing. As technology advances, new types of processors are added, such as those optimized to execute neural networks, so that Starkey can continue to improve the sound quality of our hearing aids and ability of the hearing aid wearer to understand speech in a challenging environment.

SCIF:

All Starkey Omega AI wireless hearing aids are SCIF-compliant devices. All hands-free, bi-directional wireless audio transmission features may be disabled in the hearing firmware through fitting software by the audiologist. Those restricted features cannot be activated by the patient by any means.

*For iOS users only.

**Programs accessible via hearing aid user control or wireless accessory.

Apple Watch® is a trademark of Apple, Inc., registered in the U.S. and other countries and regions. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The Auracast™ word mark and logos are trademarks owned by the Bluetooth SIG. Any use of such marks by Starkey is under license. Starkey, TeleHear, Hear Share, Multiflex Tinnitus, Edge Mode, and StarLink are registered trademarks of Starkey Laboratories, Inc. Omega is a trademark of Starkey Laboratories, Inc.

©2025 Starkey Laboratories, Inc. All Rights Reserved. 7/25 SPEC3106-00-EN-GS




Starkey | Technical Data

CIC

Completely-In-Canal

Fitting Range

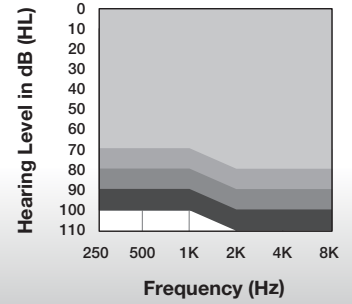

Omega AI

Color Guide

Faceplate Colors



Shell Colors



Patient Features

- IP Rating: 68
- Tinnitus Technology
- Wireless Connectivity

Omega AI Technology

- Personal health monitoring technology with embedded sensors and artificial intelligence
- Compatible with StarLink Remote Control 2.0 and StarLink Edge accessories

	L	M	P	UP	Matrices: L, M, P, UP Battery Size: 312
Measurement	ANSI/IEC 2cc Coupler	ANSI/IEC 2cc Coupler	ANSI/IEC 2cc Coupler	ANSI/IEC 2cc Coupler	
Peak OSPL90 (dB SPL)	109	113	117	125	
HFA OSPL90 (dB SPL)	104	109	113	121	
Peak Gain (dB)	42	51	60	67	
HFA Full-On Gain (dB)	36	47	54	63	
Frequency Range (Hz)	<100-9400	<100-8500	<100-6600	<100-5800	
HFA Frequencies (kHz)	1.0, 1.6, 2.5	1.0, 1.6, 2.5	1.0, 1.6, 2.5	1.0, 1.6, 2.5	
Reference Test Gain (dB)	27	32	36	44	
Equivalent Input Noise (dB SPL) <i>Expansion On (Default)</i>	15	15	15	15	
Equivalent Input Noise (dB SPL) <i>Expansion Off</i>	25	25	25	25	
Harmonic Distortion					
500 Hz (%)	<3	<3	<3	<3	
800 Hz (%)	<3	<3	<3	<3	
1600 Hz (%)	<3	<3	<3	<3	
Estimated Battery Life for 16-Hour Day					
Without Streaming (days)	Up to 8	Up to 8	Up to 8	Up to 8	
With Streaming (days)	Up to 7	Up to 7	Up to 7	Up to 7	
Battery Current (mA)	1.3	1.3	1.3	1.4	
Tinnitus Therapy Stimulus					
Max RMS Output (dB SPL)	87	87	87	87	
Weighted RMS Output Level (dB SPL)	87	87	87	87	
Max 1/3 Octave Output (dB SPL)	87	87	87	87	

L

M

P

UP







Starkey | Technical Data

CIC

Completely-In-Canal

Omega AI

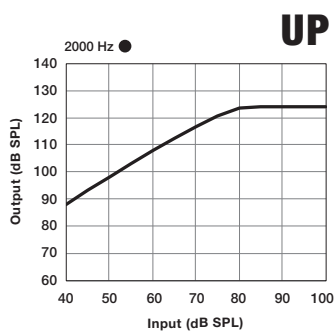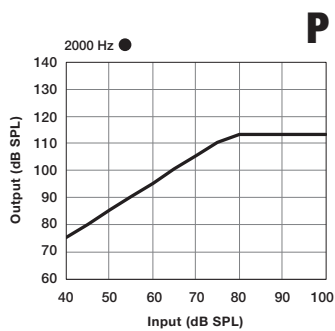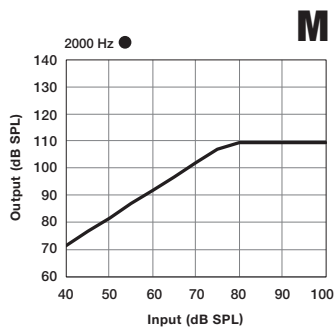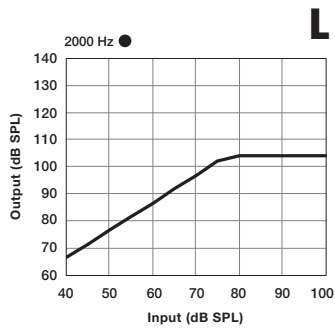

Latency (ms) 4.3

Attack Time (ms) 3

Release Time (ms) 60




Starkey | Technical Data

CIC

Completely-In-Canal

Fitting Range

● UP-

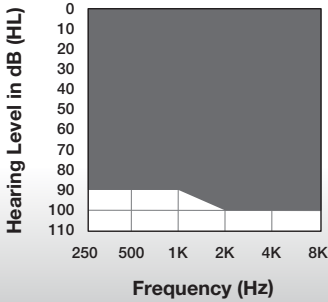

Omega AI

Color Guide

Faceplate Colors



Shell Colors



Patient Features



- IP Rating: 68
- Tinnitus Technology
- Wireless Connectivity

Omega AI Technology

- Personal health monitoring technology with embedded sensors and artificial intelligence
- Compatible with StarLink Remote Control 2.0 and StarLink Edge accessories

	UP-	Matrices: UP- Battery Size: 312
--	-----	------------------------------------

Measurement	ANSI/IEC 2cc Coupler
Peak OSPL90 (dB SPL)	119
HFA OSPL90 (dB SPL)	116
Peak Gain (dB)	65
HFA Full-On Gain (dB)	59
Frequency Range (Hz)	<100-6200
HFA Frequencies (kHz)	1.0, 1.6, 2.5
Reference Test Gain (dB)	39
Equivalent Input Noise (dB SPL) <i>Expansion On (Default)</i>	15
Equivalent Input Noise (dB SPL) <i>Expansion Off</i>	25
Harmonic Distortion	
500 Hz (%)	<3
800 Hz (%)	<3
1600 Hz (%)	<3

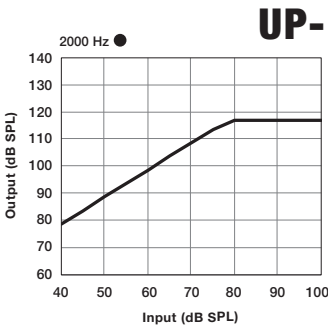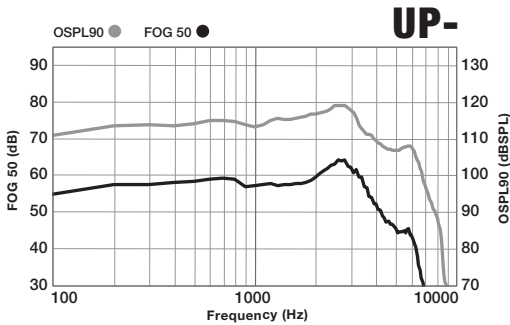

Estimated Battery Life for 16-Hour Day	
Without Streaming (days)	Up to 8
With Streaming (days)	Up to 7
Battery Current (mA)	1.3
Tinnitus Therapy Stimulus	
Max RMS Output (dB SPL)	87
Weighted RMS Output Level (dB SPL)	87
Max 1/3 Octave Output (dB SPL)	87

Latency (ms)	4.3
Attack Time (ms)	3
Release Time (ms)	60