

QUICKTIP

My Audibel

Apple Version

The image shows a smartphone screen with the My Audibel app interface. The background is a dark green color. At the top, the status bar shows the time 9:41, signal strength, Wi-Fi, and battery icons. Below the status bar, there are two battery status icons labeled 'L' and 'R'. In the top right corner, there is a microphone icon for the Smart Assistant. The main content area features the title 'Personal' and the subtitle 'Your everyday hearing setting'. Below this, there are three circular icons: 'Edge Mode+' (with a soundwave icon), 'Programs' (with a document icon), and 'My Hearing' (with a brain icon). At the bottom of the main content area, there is a binaural volume control slider. The bottom dock contains four icons: 'Home' (house icon), 'Health' (heart icon), 'Learn' (graduation cap icon), and 'More' (three horizontal lines icon). Lines connect these elements to descriptive callout boxes.

Battery Status
 Shows the battery level for the left (L) and right (R) hearing aids.

Smart Assistant
 Tap to use voice commands to change programs, set reminders, ask questions on hearing aid topics, and more.

Current Program
 Displays the current hearing program, 'Personal', and the subtitle 'Your everyday hearing setting'.

Program Options
 Access quick details on the current program, make sound adjustments, and more.

Edge Mode+
 Tap to get the best sound instantly for the current environment.

Programs
 Access programs or create a custom program.

My Hearing
 Manage hearing aid features and settings.

Binaural Volume Control
 Adjust the volume for both hearing aids simultaneously.

Return to Home
 Tap the Home icon to return to the home screen.

Health
 View progress and insights on hearing and physical activity (steps, exercise, and more).

Learn
 Access helpful how-to's, tips, and videos on hearing aid topics.

More
 Access account information, app settings, TeleHear, translations, reminders, and more.

Downloading My Audibel

You must have an Apple ID to download My Audibel. To set up an account, visit <https://appleid.apple.com>.

- 1 Once an account is set up, open the App Store.
- 2 Search for **My Audibel**.
- 3 Select **Get**.



MyAUDIBEL

Pairing

Pairing should be completed after the hearing aids have been programmed in the Pro Fit software. Ensure that Pro Fit is closed prior to pairing with any Apple device.

- 1 Select **Settings > Accessibility > Hearing Devices**
- 2 Ensure Bluetooth® setting is on.
- 3 Power off, then power on the hearing aids to place in pairing mode.
- 4 Select the hearing aids when they appear on the screen.
- 5 When prompted, select **Pair**. You will select Pair twice for a binaural set of hearing aids.
- 6 Ensure it says **“Connected”** after the hearing aid name.

My Audibel Initial Setup

- 1 Tap on **Get Started** to begin the app set-up process.
- 2 Review the Terms of use. Tap **Accept** to continue.
- 3 Ensure hearing aids are charged and turned on. Select **Continue**.
- 4 A prompt will ask if the user would like to connect to Bluetooth. Select **Ok**. My Audibel will connect to the hearing aids. Select **Continue**.
- 5 Sign in or create an account with an e-mail address and password. **An account is required**. Cloud Connectivity allows for TeleHear, Fall Alert and for app/hearing aid information to be backed up and restored.
- 6 Select **Continue** to allow My Audibel access to the user's location and to share notifications.
- 7 Users are asked to allow My Audibel to send the user notifications. Select **Don't Allow** or **Allow**.
- 8 Users are asked to allow My Audibel to collect and store Hearing aid data, Activity tracking, and Analytics. Select **Allow** or **Not Now**.
- 9 Users are asked to allow My Audibel to send the user Critical Alerts. Select **Don't Allow or Allow**. Selecting **Allow** ensures that a Fall Alert will be presented to the user even if the phone is muted or Do Not Disturb is on.

You're All Set. Select **Got it** to enter My Audibel.

Features may vary by country.



AUDIBEL