

SURFLINK MINI MOBILE

PRODUCT AVAILABILITY: Muse iQ, SoundLens Synergy iQ, Muse, and Z Series Wireless Devices

The SurfLink Mini Mobile consists of the SurfLink Remote Microphone 2 and the SurfLink Mini Mobile Adapter.

The SurfLink Mini Mobile allows users to wirelessly stream phone calls and other audio from connected Bluetooth devices. The SurfLink Mini Mobile Adapter attaches to the SurfLink Remote Microphone 2.

The SurfLink Mini Mobile Adapter adds Bluetooth phone call and audio streaming capabilities to the SurfLink Remote Microphone 2.

Prior to using the SurfLink Mini Mobile:

1. Select the check box next to SurfLink Mini Mobile on the Accessories screen in Inspire X to ensure customization of Stream Boost and hearing device microphone offsets while streaming.
2. Ensure the hearing devices have been synced to the SurfLink Remote Microphone 2. Refer to the *SurfLink Remote Microphone QuickTIP* for additional information.

SURFLINK MINI MOBILE: SETUP

1. Attach the SurfLink Mini Mobile Adapter to the female microUSB jack of the SurfLink Remote Microphone 2.
NOTE: The SurfLink Mini Mobile Adapter will only operate with the SurfLink Remote Microphone 2.
2. Ensure the SurfLink Mini Mobile Adapter and SurfLink Remote Microphone 2 are charged. The SurfLink Mini Mobile Adapter can be charged alone or both the SurfLink Mini Mobile Adapter and SurfLink Remote Microphone 2 can be charged together while attached.
3. Pair Bluetooth devices, such as cell phones, computers, or MP3 players, to the SurfLink Mini Mobile Adapter. Up to two Bluetooth devices may be paired with the SurfLink Mini Mobile Adapter, but audio will only stream from one Bluetooth device at a time.

To pair Bluetooth devices:

- » Put the Bluetooth device (i.e. cell phone or MP3 player) into pairing mode.
- » With the SurfLink Mini Mobile Adapter off, press and hold the User Control button on the SurfLink Mini Mobile Adapter for 6 seconds to enter into a pairing state.
- » Select the SurfLink Mini name on the Bluetooth device to complete the pairing.
- » After successful pairing, the SurfLink Mini Mobile Adapter will remain powered on so the audio link can easily be verified.



SurfLink Mini Mobile: Combination of SurfLink Remote Microphone 2 and SurfLink Mini Mobile Adapter




SurfLink Mini Mobile Adapter

PHONE CALLS

1. Ensure that the SurfLink Remote Microphone 2 and SurfLink Mini Mobile Adapter are attached and both are powered on.
 - » Slide the SurfLink Remote Microphone 2 to the ON position.
 - » Press and hold the user button on the SurfLink Mini Mobile Adapter for 3 seconds. The LED light will blink blue, indicating that the SurfLink Mini Mobile Adapter is on and connected to a Bluetooth device.
2. To make an outgoing phone call from a connected Bluetooth phone:
 - » Initiate a call on the connected Bluetooth phone.
 - » The call will automatically stream to the hearing devices. The SurfLink Mini Mobile Adapter will pick up the user's voice and transmit it back through the phone to the other person on the call.

NOTE: For a no-look, hands-free conversation, the user can wear the SurfLink Mini Mobile on a lanyard around their neck, clipped to their lapel, or placed near their mouth.
3. To accept an incoming phone call from a connected Bluetooth phone:
 - » The phone ringtone will play in the hearing devices.
 - » Briefly press and release the User Control button on the SurfLink Mini Mobile Adapter to accept the call.
 - » During the phone call, the SurfLink Mini Mobile Adapter will pick up the user's voice and transmit it back through the phone to the caller.
 - » To reject an incoming call from a connected Bluetooth phone, press the User Control button on the SurfLink Mini Mobile Adapter for more than 3 seconds.
4. While streaming phone calls, the hearing device microphones will follow the settings programmed within the Accessories screen.
5. To adjust the streamed phone call volume, adjust the volume on the Bluetooth device (i.e. cell phone) or adjust the volume on the hearing devices.

NOTE: When the SurfLink Remote Microphone 2 is ON and the SurfLink Mini Mobile Adapter is OFF, the system will function as a remote microphone and will not stream audio from connected Bluetooth devices.

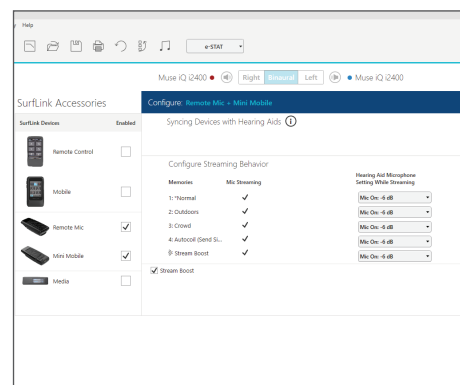

Setup In Inspire

MUSIC AND OTHER AUDIO

1. Ensure that the SurfLink Remote Microphone 2 and SurfLink Mini Mobile Adapter are attached and both are powered on. Refer to Step #1 in the *Phone Calls* section above for additional information.
2. On the connected Bluetooth device (i.e. smartphone or MP3 player), start playing music or other audio.
3. While streaming music and other audio, the hearing device microphones will follow the settings programmed in the Accessories screen in Inspire X.
4. To change the streamed audio volume, adjust the volume on the Bluetooth device (i.e. smartphone or MP3 player) or adjust the volume on the hearing devices.
5. Press stop on the media player of the connected Bluetooth device to halt streaming music or other audio.