

**QUICKTIP**

# My Audibel

Android Version

The image shows a smartphone screen with the My Audibel app interface. The screen has a dark green background. At the top, the status bar shows the time 9:41, signal strength, Wi-Fi, and battery icons. Below the status bar, there are two battery icons labeled 'L' and 'R'. In the top right corner, there is a microphone icon. The main content area features the word 'Personal' in large white text, followed by the subtitle 'Your everyday hearing setting'. Below this, there are three circular icons: 'Edge Mode+' (with a soundwave icon), 'Programs' (with a document icon), and 'My Hearing' (with a brain icon). At the bottom of the main content area, there is a binaural volume control slider with a white knob in the center. The bottom of the screen has a white navigation bar with four icons: 'Home' (house icon), 'Health' (heart icon), 'Learn' (graduation cap icon), and 'More' (three horizontal lines icon). Lines connect various parts of the screen to text boxes on the left and right, providing descriptions for each feature.

**Battery Status**

**Smart Assistant**  
Tap to use voice commands to change programs, set reminders, ask questions on hearing aid topics, and more.

**Current Program**

**Program Options**  
Access quick details on the current program, make sound adjustments, and more.

**Edge Mode+**  
Tap to get the best sound instantly for the current environment.

**My Hearing**  
Manage hearing aid features and settings.

**Programs**  
Access programs or create a custom program.

**Binaural Volume Control**

**Return to Home**

**More**  
Access account information, app settings, TeleHear, translations, reminders, and more.

**Health**  
View progress and insights on hearing and physical activity (steps, exercise, and more).

**Learn**  
Access helpful how-to's, tips, and videos on hearing aid topics.

## Downloading My Audibel

---

You must have a Google Play Store account to download My Audibel. A Google Play Store account can be created by going to **Settings>Accounts**. Choose an existing account or select **Add Account** to create a new account.

- 1 Once an account is set up, open the Play Store.
- 2 Search for **My Audibel**.
- 3 Select **Install**.



**My**AUDIBEL

## My Audibel Pairing and Initial Setup

---

My Audibel will search for and pair to the hearing aids. Pairing should be completed after the hearing aids have been programmed in the Pro Fit software. Ensure that Pro Fit is closed prior to pairing with any Android device.

- 1 Tap on **Get Started** to begin the app set-up process.
- 2 Review the Terms of use. Tap **Accept** to continue.
- 3 Ensure hearing aids are charged and turned on. Select **Continue**.
- 4 A prompt will inform the user that Nearby Devices Permission is required. Tap **Ok**, then select **Allow**.
- 5 When prompted, select **Pair**, then tap **Allow**.
- 6 My Audibel will connect to the hearing aids. Select **Continue**.
- 7 Sign in or create an account with an e-mail address and password. **An account is required**. Cloud Connectivity allows for TeleHear, Fall Alert and for app/hearing aid information to be backed up and restored.
- 8 Select **Continue** to allow My Audibel access to the user's location and to share notifications.
- 9 Users are asked to allow My Audibel to send the user notifications. Select **Allow** or **Don't Allow**.
- 10 Users are asked to allow My Audibel to collect and store Hearing aid data, Activity tracking, and Analytics. Select **Allow** or **Not Now**.

You're All Set. Select **Got it** to enter My Audibel.

*Features may vary by country.*