

## QUICKTIP

# Rechargeable Hearing Aid Chargers

The Standard and Premium Standard Chargers operate as a charging station, dry-aid kit, and case for standard lithium-ion rechargeable hearing aids. The Custom and Premium Custom Chargers operate as a charging station and case for custom lithium-ion rechargeable hearing aids.\* All chargers are equipped with front LED lights to indicate charging status for the hearing aids.

The Standard and Custom Chargers do not have an on-board battery and must be plugged into an external power source (i.e., wall outlet) to function/charge the hearing aids.

The Premium Standard and Premium Custom Chargers have an on-board battery which allows charging of the hearing aids to occur without a power source.

## LED Indicators for Hearing Aid Charging

The front of the chargers is equipped with two LED indicators to provide information about hearing aid charging.

- When the hearing aids are placed into the charger, the front LED lights will glow green to indicate successful charging.
- When the hearing aids are fully charged, the front LED lights will be solid green indicating a full charge.
- If there is an issue with charging and the hearing aids are in a fault state, the front LED lights will blink red.

**NOTE:** If a fault state is detected, remove the hearing aids from the charger, wait until the LEDs turn OFF, and re-insert the hearing aids. If the fault state continues, contact your hearing professional.



## LED Indicators for Premium Standard & Custom Charger On-board Battery

---

The back of both Premium Chargers is equipped with four LED indicators to provide information about the status of the on-board battery. The on-board battery allows charging of the hearing aids to occur while the Premium Chargers are away from a power source.



- Four solid green lights indicate the Premium Chargers are plugged into a power source and are fully charged.

**NOTE:** *When charging without the power cord, the back LEDs will turn off after 10 seconds to preserve battery life.*

- When away from a power source, four solid lights indicate the battery has more than 75% charge.
- Three solid lights indicate the battery has less than 75% charge.
- Two solid lights indicate the battery has less than 50% charge.
- One blinking light indicates that the charge is low, and the Premium Chargers should be connected to a power source to charge the on-board battery.

**NOTE:** *When plugged into a power source, both the hearing aids and on-board battery will charge simultaneously.*

## Use

---

- 1 Standard and custom hearing aids will be fully charged in 3.5 hours.

**NOTE:** *The on-board battery in the Premium Standard and Custom Chargers will be fully charged in 2.5 hours. On a full charge, hearing aids can be charged up to three times via the on-board battery without being connected to a power source.*

- 2 It is safe to keep the hearing aids in the charger as they cannot overcharge.

- 3 Charging will occur with the lid open or closed.

**NOTE:** *It is recommended to keep the Standard and Premium Standard charger lids closed to preserve the desiccant puck if also using as a dehumidifier*

- 4 Hearing aids automatically power ON when removed from the charger and power OFF when inserted into the charger.

**NOTE:** *The hearing aids will power ON in the charging case if there is no power source.*

\* Compatible with all custom lithium-ion rechargeable hearing aids. Contact your Sales Rep for specific information.

The Starkey logo is a trademark of Starkey Laboratories, Inc.

©2023 Starkey Laboratories, Inc. All Rights Reserved. 2/23 FLYR3397-03-EN-SC