









Hearing Aid Programming Guide | Programming connections



Supported Programmers:						
● NOAHLINK Wireless 2.0 / NOAHLINK Wireless / Wireless ● HiPro 2 / HiPro USB / SpeedPort 2 / SpeedPort / NOAHLINK						
Product	 RIC 312	 RIC RT*/RIC R	 mRIC R*	 2.4 GHz Custom	 Custom	 CIC R NW
Aris AI	●	● (RIC RT)	●	●	N/A	N/A
Signature Series	N/A	N/A	N/A	N/A	●	●
Vitality AI	●	● (RIC RT)	●	●	N/A	N/A

Product	 BTE R	 BTE 13	 Power Plus BTE 13
Arc AI	●	●	●

NOTES


- It is recommended that a battery be in the hearing aid when programming with a flex strip.
- VC position does not matter as the hearing aid volume control is disabled during programming.
- Audibel uses purple programming cables. Flex strips will connect to the cables.

INSTRUCTIONAL VIDEO

- [Customs Programming Flex Strip Insertion](#)

Hearing Aid Programming Guide | Cable connections

Signature Series

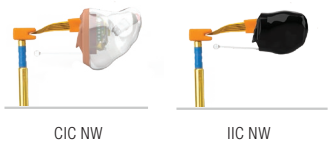

NOTE:
Programming cables and flex strips
can be ordered from AudibelPro.com